[ES] Información para el consumidor

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Ingredientes: Polidimetilsiloxanos, siloxanos, alquilmetil-silicones

[EN] Consumer Information

Strata-triz is a rapidly drying, non-sticky, transparent silicone gel formulation for the treatment of scars, both old and new.

When used as directed Strata-triz dries to form a silicone gel sheet, a protective layer that is gas permeable and waterproof which hydrates and protects scars from chemical and microbial invasion, allowing scars to normalize the collagen synthesis cycles, producing a normal mature scar.

Strata-triz is used to soften and flatten scars, relieve the itching and discomfort as well as to reduce redness and discoloration associated with scars.

Strata-triz can be easily applied to all areas of the skin, including exposed areas like the face and neck as well as joints and hairy areas without the need for shaving.

Strata-triz was developed for use on all types of scars including those resulting from:

- general and cosmetic surgery
- trauma
- chronic wounds
- burns
- bites
- acne
- and other diseases of the skin

The Updated International Clinical Recommendations on Scar Management confirmed that silicone gel should be used as first line therapy in the initial management of scars and in the prevention of hypertrophic scars and keloids.*

Intended use

Strata-triz is used for the prevention and treatment of keloids and hypertrophic scars.

Strata-triz can also be used in conjunction with other invasive scar treatment options to improve overall results, including surgical excision and resuturing, intrasurgical injections and pressure garments.

Strata-triz is suitable for children and people with sensitive skin.

Strata-triz contains no alcohol, parabens or fragrances.

Strata-triz may be used on infants, during pregnancy and while breastfeeding.

Directions for use

On first use of 5g, 10g and 20g tubes remove the cap, cut the nozzle tip of the tube and close with the cap after use.

On 50g tubes remove the cap, then the protective seal and close with the cap after use.

Ensure that the scar is clean and dry.

Apply a very thin layer of Strata-triz to the scar and allow to dry.

When applied correctly, Strata-triz should be dry in 3–4 minutes. If it takes longer to dry you have probably applied too much. Gently remove the excess with a tissue and allow the drying process to continue.

Once dry, Strata-triz can be covered with sunscreen, cosmetics, pressure garments or casts.

Strata-triz should be applied once daily, or twice daily to exposed areas or as advised by your physician.

For best results Strata-triz should maintain continuous contact with the skin (24 hours a day/7 days a week).

Recommended duration of treatment

Strata-triz is recommended as a minimum treatment of 60–90 days (24 hours a day/7 days a week). Larger and older scars may take longer.

Continued use is recommended until no further improvement is seen.

How much Strata-triz do I need?

Strata-triz is an advanced formulation that requires less product per application than typical creams or gels.

- **Strata-triz 5 g** is enough to treat a 2–3 cm linear scar for over 90 days.
- **Strata-triz 10 g** is enough to treat a 5–7 cm linear scar for over 90 days.

- **Strata-triz 20 g** is enough to treat a 10–14 cm linear scar for over 90 days.
- **Strata-triz 50 g** is enough to treat an area of 6 × 12 cm for over 90 days.

Caution

- For external use only.
- Strata-triz should not be applied on third degree burns or to open wounds.
- Strata-triz should not be placed in contact with mucous membranes or the eyes.
- Strata-triz should not be applied over other skin treatments without the advice of your physician.
- Strata-triz may stain clothing if not completely dry. If staining occurs, dry cleaning should be able to remove it without damaging the fabric.
- For correct storage please reclose the tube tightly with the cap.
- If irritation occurs, discontinue use and consult your physician.
- Keep out of the reach of children.
- Do not use after the expiration (EXP) date printed on the tube.
- Strata-triz does not require special disposal methods. Please follow your local disposal regulations.
- Within the European Union, serious incidents related to the use of Strata-triz should be reported to Stratpharma and the respective competent authority in the country the user is established.

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Stratpharma AG
Stratpharma Limited
Freyr Life Sciences GmbH